


The packaging of this cushion is sealed prior to despatch and the cushion itself will be clean prior to closure. If the cushion had become exposed or damaged during transit, please contact us at the address below.

# Instruction Guide

## Memory-Visco Cushion

3LMC43

Foam cushion with top and bottom surface memory foam. Two way stretch, vapour permeable removable cover with sewn seams. Zipped closure to two sides with waterfall flap.

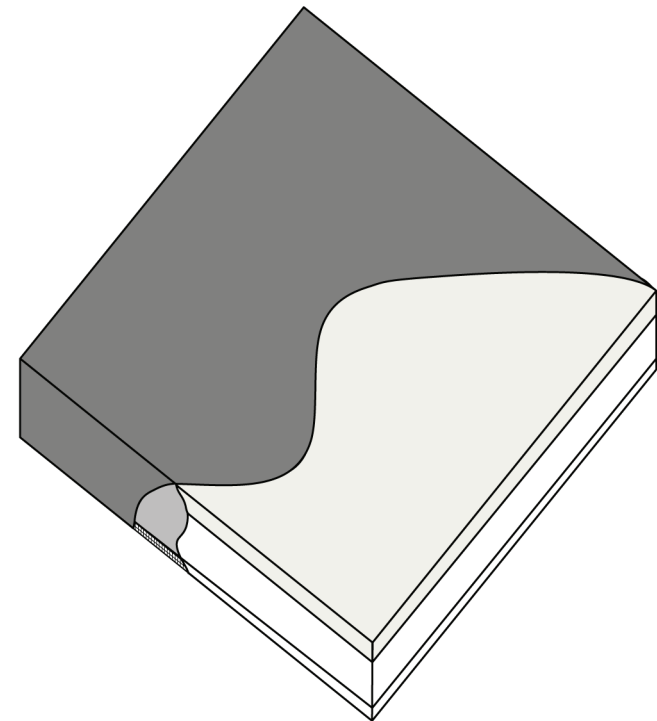

Thorpe Mill Ltd  
Unit 1 a Aireside Business  
Park, Royd Ings Ave  
Keighley BD21 4BZ  
Tel : 01535 682 630

### **User Instructions**

The Memory-Visco Cushion has been designed for use in wheelchairs and/or day care chairs by patients in the medium to high risk category\* with a weight guide of up to 127Kg/ 20 stones.

Ensure that the surface of the chair seat is clear from debris before placing the cushion – debris can damaged the coating of the fabric.

The label on this cushion has a unique batch number which should be used in the event of a warranty claim. NEVER remove his label.

### **Removing the Cover**

The cover on this cushion is removable for cleaning purposes. Always ensure that, if removed, the cover is returned to the same inner. Using this cover on a different cushion core MAY render the warranty invalid.

The flap covers a zip on two sides. This zip should be positioned to the sides and rear of the cushion.

Following the above instructions will ensure that there is no front seam other than that at the lower edge of the cushion.

### **Cleaning**

For general cleaning and superficial dirt, use a disposable wipe with a solution of warm water and neutral detergent. Do not use abrasive or phenolic-based cleaners. Do not mangle or iron.

To disinfect in situ, use 0.1% sodium hypochloride solution (1,000 ppm chlorine).

Removable, contaminated covers should be laundered at 75C for not less the 10 minutes or at 95C for not less than 3 minutes. Do not mangle or iron. Spin and tumble- dry at not more than 60C. Thoroughly rinse and dry the cover before reuse or storage. Store in a cool dry area.

### **Storage & Care**

Store the cushion flat – do not fold

Do not place heavy objects on the cushion surface when not in use

Do not allow sharp objects to penetrate the cover coating

Keep the cushion clean

Regularly inspect the cushion for visible soiling or damage

Do not store in damp conditions

### **Warranty**

Covered against faulty manufacture and materials as replacements goods. Normal wear and tear expected would not be included.

### **Flammability**

The Memory-Visco Cushion meets the requirements of BS6807:2006 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) regulations 1988.

\*Indication of use based upon interface pressure readings-not a prescriptive criteria. Always defer to clinician's judgement.